Medicontur Orvostechnikai Kft.

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E-mail: orders@coloron.eu

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(EN) INSTRUCTIONS FOR USE

1. **COLORON® LuxBlue** blue-light filter base lens – Filter Category 0.
2. **COLORON® LuxDim** color vision enhancement lens for low-light, semi-shaded conditions inserted in the 1st clip – Filter Category 2.
3. **COLORON® LuxBright** color vision enhancement lens for medium-high-light conditions inserted in the 2nd clip – Filter Category 3.

PRODUCT DESCRIPTION: COLORON® glasses are assembled by Medicontur Medical Engineering Ltd., a company with more than 30 years of experience in the manufacture of high precision optical systems in the medical field of ophthalmology. All optics undergo a thorough treatment process and block 100% UVA/UVB/UVC light. Anti-scratch coating is applied to protect the optic for long term use. A hard-case, a microfiber cleaning cloth and a user guide are included in the product packaging. *

INTENDED USE: The COLORON® LuxBright and LuxDim lenses and eyewear series are intended for use by people with red-green color vision deficiency to improve color discrimination and overall color perception in bright and dim lighting conditions. COLORON® eyewear are assistive devices and are not a cure for color vision deficiency. The plano LuxBlue blue-light lenses, also known as computer or screen lenses protect against overexposure to blue light and can be

worn while working on digital screens. Optionally, the blue light lenses can be exchanged with prescription optics by your optician.

INSTRUCTIONS: For first time wear, allow at least 10 minutes for your eyes to adapt. During the adjustment period please do not remove the glasses to ensure effectiveness. Perceived results vary by the type and severity of color vision deficiency – For best results use the eyewear frequently and in bright and colorful environments.

CARE: For consistent visual quality clean the lenses regularly with mild dish soap and warm water, rinse it with clean water and dry it with the provided micro fiber cleaning cloth. Use the hardcase for transporting the eyewear to ensure protection from external forces. *

STORAGE: Store COLORON® eyewear in cool, dry areas away from extreme temperatures as it can damage the lenses.

LIMITATION OF USE: The lens is not certified for use in occupations where color blind individuals may be restricted and should not be used to take color blindness screening tests to qualify for such occupations. The eyewear is not designed to protect the wearer from injury and is not intended for use as protective equipment.

CAUTION:  According to the spectral transmittance requirements of ANSI Z80.3-2018 Section 4.10.2.3 and ISO 12312-1:2022 Section 5.3.2, The LuxBright and LuxDim lenses are not suitable for driving and road use. Tinted lenses may impair visibility in low light conditions and should not be used for driving in twilight, night, or through tunnels.

Sunglasses are not suitable for direct observation of the sun, or for viewing solar eclipses. If the eyewear causes headache, nausea, or impairs your vision, discontinue use of the eyewear.

WARRANTY: A lens with a diopter that fits into the base frame should be selected so that the lens does not protrude from the plane of the base frame, thus avoiding loss of functionality of the spectacles and accidents from use. In case of incorrect lens selection, the manufacturer cannot be held responsible for any damage caused.

All COLORON® glasses are warranted against manufacturing defects from the date of purchase. To make a warranty claim, you will need a valid proof of purchase. If you would like to make a warranty claim for your COLORON® glasses, please contact our Customer Support Department via email:

returns@coloron.eu. For more information, please check our warranty policy at www.coloron.eu/warranty.

CONFORMANCE:  COLORON® Eyewear comply with Regulation (EU) 2016/425, EN ISO 12132-1, EN ISO 14889. Registration Number: HU-2021/02722-01691. Declaration of conformity can be found here: www.coloron.eu/publications.

INTELLECTUAL PROPERTY: COLORON® and the eye emblem are registered trademarks of Medicontur Medical Engineering Ltd.

** Our Harry product is not placed in a hard case, but in a microfibre pouch.*

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