**Medicontur Ltd.**

Herceghalmi út 1.  
Zsámbék, 2072  
Hungary

Production site:

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1. B2B instructions for use (EN)

**COLORON® Color vision enhancement lenses**

1. **COLORON® Xperience, Xperience Reflex, Strong, Strong Reflex** color vision enhancement lens for low-light, semi-shaded conditions – Filter Category 2.
2. **COLORON® Sun, Sun Reflex** color vision enhancement lens for medium-high-light conditions – Filter Category 3.
3. **COLORON® Mild** color vision enhancement lens for low-light conditions – Filter Category 1.

**COLORON® Color vision enhancement 3-in-1 Eyewear set**

1. **COLORON® Strong, Strong Reflex, Xperience, Xperience Reflex** color vision enhancement lenses for low-light, semi-shaded conditions inserted in clip – Filter Category 2.
2. COLORON® 3-in-1 frame
3. COLORON® Hard case
4. COLORON® Microfibre cloth
5. COLORON® Microfibre pouch
6. COLORON® Carton box

**COLORON® Color vision enhancement Sunglasses**

1. **COLORON® Sun, Sun Reflex** color vision enhancement lenses for medium-high-light conditions – Filter Category 3.
2. COLORON® Sunglasses frame
3. COLORON® Hard case
4. COLORON® Microfibre cloth
5. COLORON® Microfibre pouch
6. COLORON® Carton box

**COLORON® Indoor color vision enhancement glasses**

1. **COLORON® Xperience, Xperience Reflex, Strong, Strong Reflex** color vision enhancement lenses for low-light, semi-shaded conditions – Filter Category 2 or **COLORON® Mild** color vision enhancement lens for low-light conditions – Filter Category 1.
2. COLORON® Indoor frame
3. COLORON® Hard case
4. COLORON® Microfibre cloth
5. COLORON® Microfibre pouch
6. COLORON® Carton box\*

#### **COLORON® Base frame**

1. COLORON® Base frame – with or without blue-light filter base lens
2. COLORON® Hard case
3. COLORON® Microfibre cloth
4. COLORON® Microfibre pouch
5. COLORON® Carton box

#### **COLORON® Color vision enhancement clip and flip clip**

1. **COLORON® Sun, Sun Reflex** color vision enhancement lenses for medium-high-light conditions – Filter Category 3 or **COLORON® Xperience, Xperience Reflex, Strong, Strong Reflex** color vision enhancement lenses for low-light, semi-shaded conditions – Filter Category 2 or **COLORON® Mild** color vision enhancement lens for low-light conditions – Filter Category 1.
2. COLORON® Microfibre pouch
3. COLORON® Clip box

**PRODUCT DESCRIPTION:** All optics undergo a thorough treatment process and block UVA/UVB/UVC light. anti-scratch coating is applied to protect the optic for long term use. COLORON® glasses are assembled by Medicontur Medical Engineering Ltd., a company with more than 30 years of experience in the manufacture of high precision optical systems in the medical field of ophthalmology.

**INTENDED USE:** The COLORON® Sun, Sun Reflex, Xperience, Xperience Reflex, Strong, Strong Reflex, Mild lenses and eyewear series are intended for use by people with red-green color vision deficiency to improve color discrimination and overall color perception in bright and dim lighting conditions. COLORON® eyewear are assistive devices and are not a cure for color vision deficiency.

**INSTRUCTIONS:** For first time wear, allow at least 10 minutes for your eyes to adapt. For faster color adaptation, always close your eyes when putting on the glasses and open them when the glasses are on your face. During the adjustment period please do not remove the glasses to ensure effectiveness.

When you put on the glasses, all objects will first appear red/pink. This is due to the eye's natural adaptation process. The new and improved color balance is achieved after about 5-10 minutes. Once an originally white wall or a white sheet of paper appears white again, the eye has adapted to the lenses.

The perceived result varies depending on the type and severity of the color vision deficiency. The effect of color vision enhancement glasses increases gradually. Maximum improvement is achieved after a few months of use - use glasses frequently in bright and colorful environments for best results.

**CARE:** For consistent visual quality clean the lenses regularly with mild dish soap and warm water, rinse it with clean water and dry it with the provided microfibre cleaning cloth. Use the hardcase for transporting the eyewear to ensure protection from external forces. \*

**STORAGE:** Store COLORON® eyewear in cool, dry areas away from extreme temperatures as it can damage the lenses.

**LIMITATION OF USE:** The lens is not certified for use in occupations where colorblind individuals may be restricted and should not be used to take color blindness screening tests to qualify for such occupations. The eyewear is not designed to protect the wearer from injury and is not intended for use as protective equipment.

#### **CAUTION:**

Driving: the Xperience, Xperience Reflex, Sun, Sun Reflex, Mild products may be used for road use and driving in terms of ISO 12312-1:2022, chapter 5.3.2. Their use will comply with the criteria specified in this standard. Medicontur Ltd. shall not be liable for any damages or accidents that may occur during the use of vehicle. User uses the product at their own risk. By using the product(s), User declares that he/she understands and accepts these terms and conditions.

Driving  : Strong, Strong Reflex products are not to be used for driving a vehicle in accordance with ISO 12312-1:2022, chapter 5.3.2.

Medicontur Ltd. does not accept any liability for any damage or accidents that may occur during the use of the vehicle. Users use the product at their own risk. By using the product, users declare that they understand and accept these terms and conditions.

Medical examinations and other medical and health tests: Medicontur Ltd. shall not be liable for the successful or unsuccessful performance of (medical) examinations or other medical and health tests performed with the product(s) or for any damages or accidents that may occur after the test or examination has been completed. By using the product(s), User declares that he/she understands and accepts these terms and conditions.

Despite Medicontur Ltd's high level of quality assurance, there may be minimal color differences between lenses, but this does not affect the performance of the lenses. When selecting lenses, please ensure that there is no difference in color between lenses that are placed in the same frame.

Tinted lenses may impair visibility in low light conditions and should not be used for driving in twilight, night, or through tunnels.

Sunglasses are not suitable for direct observation of the Sun or for observing solar eclipses. If wearing the lenses causes headaches, nausea or visual impairment, discontinue use.

**WARRANTY:** A lens with a diopter that fits into the base frame should be selected so that the lens does not protrude from the plane of the base frame, thus avoiding loss of functionality of the spectacles and accidents from use. In case of incorrect lens selection, the manufacturer cannot be held responsible for any damage caused.

All COLORON® glasses are warranted against manufacturing defects from the date of purchase. To make a warranty claim, you will need a valid proof of purchase. If you would like to make a warranty claim for your COLORON® glasses, please contact our Customer Support Department via email: [returns@coloron.eu](mailto:returns@coloron.eu).

**CONFORMANCE:** COLORON® Eyewear comply with Regulation (EU) 2016/425, EN ISO 12132-1, EN ISO 14889. Registration Number: HU-2021/02722-01691. Declaration of conformity can be found here: [www.coloron.eu/publications](http://www.coloron.eu/publications).

**INTELLECTUAL PROPERTY:** COLORON® and the eye emblem are registered trademarks of Medicontur Medical Engineering Ltd.

\* Our Harry product is not placed in a hard case, but in a microfibre pouch.

Tab 1.		
<b>0</b> TV 80-100%		
<b>1</b> TV 43-80%		
<b>2</b> TV 18-43%		
<b>3</b> TV 8-18%		

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### 3. Webshop 3in1 instructions for use

#### **(EN) INSTRUCTIONS FOR USE**

1. **COLORON® LuxBlue** blue-light filter base lens – Filter Category 0.
2. **COLORON® Xperience, Xperience Reflex, Strong, Strong Reflex** color vision enhancement lenses for low-light, semi-shaded conditions inserted in clip – Filter Category 2.
3. **COLORON® Sun, Sun Reflex** color vision enhancement lenses for medium-high-light conditions inserted in clip – Filter Category 3.

**PRODUCT DESCRIPTION:** COLORON® glasses are assembled by Medicontur Medical Engineering Ltd., a company with more than 30 years of experience in the manufacture of high precision optical systems in the medical field of ophthalmology. All optics undergo a thorough treatment process and block UVA/UVB/UVC light. Anti-scratch coating is applied to protect the optic for long term use. A hard-case, a microfiber cleaning cloth and a user guide are included in the product packaging.

**INTENDED USE:** The COLORON® Sun, Sun Reflex, Xperience, Xperience Reflex, Strong, Strong Reflex lenses and eyewear series are intended for use by people with red-green color vision deficiency to improve color discrimination and overall color perception in bright and dim lighting conditions. COLORON® eyewear are assistive devices and are not a cure for color vision deficiency. The plano LuxBlue blue-light lenses, also known as computer or screen lenses protect against overexposure to blue light and can be worn while working on digital screens. Optionally, the blue light lenses can be exchanged with prescription optics by your optician.

**INSTRUCTIONS:** For first time wear, allow at least 10 minutes for your eyes to adapt. During the adjustment period please do not remove the glasses to ensure effectiveness. Perceived results vary by the type and severity of color vision deficiency – For best results use the eyewear frequently and in bright and colorful environments.

**CARE:** For consistent visual quality clean the lenses regularly with mild dish soap and warm water, rinse it with clean water and dry it with the provided micro fiber cleaning cloth. Use the hardcase for transporting the eyewear to ensure protection from external forces.

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**LIMITATION OF USE:** The lens is not certified for use in occupations where color blind individuals may be restricted and should not be used to take color blindness screening tests to qualify for such occupations. The eyewear is not designed to protect the wearer from injury and is not intended for use as protective equipment.

**CAUTION:**

Driving: the Xperience, Xperience Reflex, Sun, Sun Reflex products may be used for road use and driving in terms of ISO 12312-1:2022, chapter 5.3.2. Their use will comply with the criteria specified in this standard. Medicontur Ltd. shall not be liable for any damages or accidents that may occur during the use of vehicle. User uses the product at their own risk.

By using the product(s), User declares that he/she understands and accepts these terms and conditions.

Driving : Strong, Strong Reflex products are not to be used for driving a vehicle in accordance with ISO 12312-1:2022, chapter 5.3.2.

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Sunglasses are not suitable for direct observation of the sun, or for viewing solar eclipses. If the eyewear causes headache, nausea, or impairs your vision, discontinue use of the eyewear.

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All COLORON® glasses are warranted against manufacturing defects from the date of purchase. To make a warranty claim, you will need a valid proof of

purchase. If you would like to make a warranty claim for your COLORON® glasses, please contact our Customer Support Department via email: [returns@coloron.eu](mailto:returns@coloron.eu). For more information, please check our warranty policy at [www.coloron.eu/warranty](http://www.coloron.eu/warranty).

**CONFORMANCE:**  COLORON® Eyewear comply with Regulation (EU) 2016/425, EN ISO 12132-1, EN ISO 14889. Registration Number: HU-2021/02722-01691. Declaration of conformity can be found here: [www.coloron.eu/publications](http://www.coloron.eu/publications).

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CON\_e-IFU\_EN,DE,FR,ES,IT,HU,NL,SV,PL\_V06  
Date of issue: 2025.08.15.

#### 4. Webshop Sunglasses instructions for use

### (EN) INSTURCTIONS FOR USE

**COLORON® Sun, Sun Reflex** color vision correction lenses for medium-high-light conditions – Filter Category 3.

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**INTENDED USE:** The COLORON® Sun, Sun Reflex lenses and eyewear series are intended for use by people with red-green color vision deficiency to improve color discrimination and overall color perception in bright lighting conditions. COLORON® eyewear are assistive devices and are not a cure for color vision deficiency.

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**CAUTION:**

Driving: the Sun, Sun Reflex products may be used for road use and driving in terms of ISO 12312-1:2022, chapter 5.3.2. Their use will comply with the criteria specified in this standard. Medicontur Ltd. shall not be liable for any damages or accidents that may occur during the use of vehicle. User uses the product at their own risk.

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CONS\_IFU\_EN,DE,FR,ES,IT,HU,NL,SV,PL\_V06  
Date of issue: 2025.08.15.

## 5. Webshop Indoor glasses instructions for use

### (EN) INSTRUCTIONS FOR USE

**COLORON® Xperience, Xperience Reflex, Strong, Strong Reflex** color vision correction lenses for low-light, semi-shaded conditions – Filter Category 2 or **COLORON® Mild** color vision correction lenses for low-light conditions – Filter Category 1.

**PRODUCT DESCRIPTION:** COLORON® glasses are assembled by Medicontur Medical Engineering Ltd., a company with more than 30 years of experience in the manufacture of high precision optical systems in the medical field of ophthalmology. All optics undergo a thorough treatment process and block UVA/UVB/UVC light. Anti-scratch coating is applied to protect the optic for long term use. A hard-case, a microfiber cleaning cloth and a user guide are included in the product packaging. \*

**INTENDED USE:** The COLORON® Xperience, Xperience Reflex, Strong, Strong Reflex, Mild lenses and eyewear series are intended for use by people with red-green color vision deficiency to improve color discrimination and overall color perception in low lighting conditions. COLORON® eyewear are assistive devices and are not a cure for color vision deficiency.

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**CARE:** For consistent visual quality clean the lenses regularly with mild dish soap and warm water, rinse it with clean water and dry it with the provided micro fiber cleaning cloth. Use the hardcase for transporting the eyewear to ensure protection from external forces. \*

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**CAUTION:**

Driving: the Xperience, Xperience Reflex, Mild products may be used for road use and driving in terms of ISO 12312-1:2022, chapter 5.3.2. Their use will comply with the criteria specified in this standard. Medicontur Ltd. shall not be liable for any damages or accidents that may occur during the use of vehicle. User uses the product at their own risk.

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The glasses are not suitable for direct observation of the sun, or for viewing solar eclipses. If the eyewear causes headache, nausea, or impairs your vision, discontinue use of the eyewear.

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*\* Our Harry product is not placed in a hard case, but in a microfibre pouch.*

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CONI\_IFU\_EN,DE,FR,ES,IT,HU,NL,SV,PL\_V06  
Date of issue: 2025.08.15.

## 6. Webshop Clip and Flip Clip instructions for use

### (EN) INSTURCTIONS FOR USE

**COLORON® Sun, Sun Reflex** color vision correction lenses for medium-high-light conditions inserted in clip – Filter Category 3 or **COLORON® Xperience, Xperience Reflex, Strong, Strong Reflex** color vision enhancement lenses for low-light, semi-shaded conditions inserted in clip – Filter Category 2 or **COLORON® Mild** color vision correction lenses for low-light conditions – Filter Category 1.

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**INTENDED USE:** The COLORON® Sun, Sun Reflex, Strong, Strong Reflex, Xperience, Xperience Reflex, Mild lenses and eyewear series are intended for use by people with red-green color vision deficiency to improve color discrimination and overall color perception in bright lighting conditions. COLORON® eyewear are assistive devices and are not a cure for color vision deficiency.

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**LIMITATION OF USE:** The lens is not certified for use in occupations where color blind individuals may be restricted and should not be used to take color blindness screening tests to qualify for such occupations. The eyewear is not designed to

protect the wearer from injury and is not intended for use as protective equipment.

**CAUTION:**

Driving: the Xperience, Xperience Reflex, Sun, Sun Reflex, Mild products may be used for road use and driving in terms of ISO 12312-1:2022, chapter 5.3.2. Their use will comply with the criteria specified in this standard. Medicontur Ltd. shall not be liable for any damages or accidents that may occur during the use of vehicle. User uses the product at their own risk.

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**WARRANTY:** All COLORON® glasses are warranted against manufacturing defects from the date of purchase. To make a warranty claim, you will need a valid proof of purchase. If you would like to make a warranty claim for your COLORON® glasses, please contact our Customer Support Department via email: [returns@coloron.eu](mailto:returns@coloron.eu). For more information, please check our warranty policy at [www.coloron.eu/warranty](http://www.coloron.eu/warranty).

**CONFORMANCE:**  COLORON® Eyewear comply with Regulation (EU) 2016/425, EN ISO 12132-1, EN ISO 14889. Registration Number: HU-2021/02722-01691. Declaration of conformity can be found here: [www.coloron.eu/publications](http://www.coloron.eu/publications).

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